

MD-Cleanser 17% EDTA Solution

INSTRUCTION FOR USE

Easier Filing with Lubricating Action

Presence of calcifications in the canal, too narrow canals in posterior teeth, removal of organic and bacteria in root canal, and smear layer removal.

Application of MD-Cleanser

For Biomechanical Preparation of the canal Apply MD-Cleanser by depositing a few drops into the root canal using a syringe, and then carefully pump the solution into the canal with a fine root canal instrument.

Continue instrumentation of the canal while being soaked in the solution.

Re-apply MD-Cleanser when necessary until canal shaping and cleaning is complete. Remove MD-Cleanser from the canal by irrigation.

For Smear Layer Removal

Irrigate the canal with MD-Cleanser, followed immediately by thorough irrigation with Sodium Hypochlorite to rinse off MD-Cleanser. Aspirate excess fluid and dry the canal.

PROPERTIES

- MD-Cleanser functions by forming a calcium-chelate solution with the calcium ions of dentin. It makes the dentin brittle and easier to instrument, and produces the cleanest dentinal walls after smear layer.

SPECIAL PRECAUTIONS FOR USE

- Use MD-Cleanser with discretion to prevent perforation. Avoid extrusion of the solution into the apex. Keep tightly capped at all times. For dental use only. Keep out of children's reach.

STORAGE/COMPONENTS

- Store in 10°C~25°C(50°F~77°F) temperature
- 100ml bottle, 1 syringe, 3 flexible tips

SHELF LIFE

- 2 years from the manufactured date.

SYMBOLS



Use By



Batch Code



0120



Attention, See instruction for use



Manufacturer

Made in Korea

Manufactured by
META BIOMED CO.LTD.

634, Yeonje-ri, Gang-ae-myeon, Cheong-won-gun
Chungcheongbuk-do, Korea
Tel: +82-43-216-0433
Fax: +82-43-217-1988
E-mail: info@meta-biomed.com
Web site: http://www.meta-biomed.com



DM-13-5

CHECKING SHEET

MD - Cleanser

TITLE	
CODE	DM-13-5
SIZE	155x162mm
PAPER	60g 모조
MAIN COLOR	INDIGO 2767C 별색 단도
SPECIAL	
FUNCTION	
PRINT COLOR	



DATE	2009. 07. 10.
DESIGNER	진희남
RECOGNITION	김상일

CONFIRMATION	META BIOMED CO.LTD.	
DATE	NAME	SIGN

2011. 06. 02 → DM-13-5 / 주소변경 (홍선미)