


MD-ChelCream 19% EDTA Cream

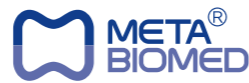

INSTRUCTION FOR USE

Easier Filing with Lubricating Action

APPLICATIONS

- Root canal preparation with files and reamers
- Presence of calcification in the canal
- Too narrow canals in posterior teeth
- Smear layer removal

MODE OF ACTION

- Removal of inorganic substances by chelation (EDTA - Ca)
- Easier filling due its lubricating action
- Produces the cleanest canal wall after smear layer removal

COMPONENTS

- MD-ChelCream 7.0g (5.4ml) x 2
- Disposable tips
- Net weight : 14g

DIRECTIONS FOR USE

1. Deposit a small amount of MD-ChelCream in the canal orifice
2. Carefully pump into the canal with a fine file or reamer
3. Re-apply MD-ChelCream when necessary or until cleaning and shaping is complete

How to use Rotary Ring

- If contact with air, as content could be hardened, so it is necessary to cover a cap on the needle after use
- Putting a Syringe into a Rotary Ring os more convenient for use to handle & Use Rotary Ring as a wrench to tighten or loosen a tip

CAUTIONS

- Use MD-ChelCream with discretion to prevent perforation
- Avoid extrusion of the solution into the apex
- Keep the syringe capped all the time

STORAGE

- Store at temperatures 18°C~24°C (64°F~75°F)

SHELF LIFE

- 2 years from the manufactured date.

SYMBOLS

- Use By
- Batch Code
- CE-Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC
- Attention, See instruction for use
- Manufacturer

Made in Korea

Manufactured by
META BIOMED CO.LTD.

634, Yeonje-ri, Gang-oe-myeon, Cheong-won-gun
Chungcheongbuk-do, Korea
Tel: +82-43-216-0433
Fax: +82-43-217-1988
E-mail: info@meta-biomed.com
Web site: http://www.meta-biomed.com

DM-16-7

CHECKING SHEET

MD - ChelCream

TITLE	
CODE	DM-16-7
SIZE	155x162mm
PAPER	60g 무조
MAIN COLOR	INDIGO 2767C 별색 단도
SPECIAL	
FUNCTION	
PRINT COLOR	



DATE	2010. 06. 10
DESIGNER	권은정
RECOGNITION	
CONFIRMATION	META BIOMED CO.LTD.
	DATE NAME SIGN

2011. 06. 02. → DM-16-7 주소변경 (홍선미)