

Meta P&Bond 5th Generation Bobbing System

This bonding agent is one step for priming and bonding

Meta P&Bond adhesive system is wet bond adhesive system that is activated by visible light curing. This bonding agent is one step for priming and bonding

INDICATIONS

- Bonding for direct composite restoration
- Bonding for Porcelain, composite, metal repair, set amalgam, post
- Bonding for porcelain veneer
- Bonding for amalgam and indirect bonding procedures with Metacem adhesive resin cement

INSTRUCTIONS

- ① Isolation: Rubber dam is the preferred method for isolation.
- ② Cavity preparation: prepare cavity with minimal tooth reduction.
- ③ Tooth conditioning /Dentin pretreatment: Apply 37% Meta Etchant gel to available enamel/dentin for 15 seconds.
- ④ Rinse and Blot dry: Remove gel with aspirator tube and rinse for 20 seconds. Blot dry conditioned area with a moist cotton pellet. Blot dry to keep moist, do not rub.
- ⑤ Bonding: Apply a double coat of Meta P&Bond by using microbrush. Gently air dry for 5 seconds. Light cure for 10 seconds.
- ⑥ Restoration or cementation.

CONTRAINDICATION

- Meta P&Bond should not be used if patients are known to be allergic to any of the ingredients

SIDE EFFECTS

- None known to date

CAUTIONS

- Avoid contact of Meta P&Bond with the skin / mucous membrane, or eyes.
- Unpolymerized Meta P&Bond may cause slightly, reversible irritation or lead to a general sensitisation against methacrylates.
- Keep away from child.
- For dental use only.
- Do not use Meta P&Bond after the date of expiration.

INSTRUCTION FOR USE

STORAGE

- Close bottle immediately after use to minimize evaporation
- Exposure to light may lead to premature polymerization
- Store at 18°C~24°C(64°F~75°F)

COMPONENT

- Meta P&Bond 5ml/g in bottle
- Microbrush
- Instruction for use

SHELF LIFE

- 2 years from the manufactured date.

SYMBOLS

- Use By
- Batch Code
- CE-Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC
- Attention, See instruction for use
- Manufacturer

Made in Korea

Manufactured by
META BIOMED CO.LTD.

634, Yeonje-ri, Gang-oe-myeon, Cheong-won-gun
Chungcheongbuk-do, Korea
Tel: +82-43-216-0433
Fax: +82-43-217-1988
E-mail: info@meta-biomed.com
Web site: http://www.meta-biomed.com



DM-25-3

CHECKING SHEET

Meta P&Bond

TITLE	
CODE	DM-25-3
SIZE	155x194mm
PAPER	60g 모조
MAIN COLOR	INDIGO 2767C 별색 단도
SPECIAL	
FUNCTION	
PRINT COLOR	



DATE	2009. 07. 10.
DESIGNER	진희남
RECOGNITION	김상일

CONFIRMATION	META BIOMED CO.LTD.	
DATE	NAME	SIGN

2011. 06. 02. → DM-25-3 / 주소변경 (홍선미)