

# Metapex Calcium Hydroxide with Iodoform

## INSTRUCTION FOR USE

### Calcium Hydroxide with Iodoform

#### APPLICATIONS

- Exposed pulp in pulp capping and pulpotomy
- Leakage canal
- Apexification
- Formation of hard tissue barrier
- Root canal filling materia

#### DIRECTIONS FOR USE

- Clean and dry it, after preparing the root canal
- Set up a disposable tip in syringe and insert the ring provided for an easy re-direction of the tip
- Disinfect the disposable tip with ethanol. Fill the canal completely by pushing the syringe plunger while withdrawing the tip slowly
- Clean any excess paste with a sterilized cotton pellet
- Remove the used tip, fix a new one and cover up

#### How to use Rotary Ring

- If contact with air, as content could be hardened, so it is necessary to cover a cap on the needle after use
- Putting a Syringe into a Rotary Ring os more convenient for use to handle & Use Rotary Ring as a wrench to tighten or loosen a tip

#### ADVANTAGES

- Easy cleaning and removal
- Excellent antibacterial effect and radiopacity
- Premixed paste in a convenient syringe
- Excellent accessibility to the root canal and prevention of cross-contamination

#### CONTENTS

- 2.2g paste in a syringe
- Disposable tips
- One ring rotator for the direction control of the tip



#### STORAGE

- Do not fill beyond the apex
- For dental use only
- Do not use in allergic person
- Keep the syringe capped at all times, when not in use
- Store in a dry and cool area to avoid dryness or separation of the paste

#### SHELF LIFE

- 2 years from the manufacturing date


#### SYMBOLS

- Use By
- Batch Code
- Attention, See instruction for use
- Manufacturer

Made in Korea

Manufactured by  
**META BIOMED CO. LTD.**

634, Yeonje-ri, Gangoe-myeon  
Cheongwon-gun, Chungcheongbuk-do, Korea  
Tel: +82-43-216-0433  
Fax: +82-43-217-1988  
E-mail: info@meta-biomed.com  
Web site: http://www.meta-biomed.com

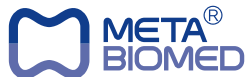

DM-09-5

|                |                    |              |              |
|----------------|--------------------|--------------|--------------|
| 품명             | Metapex 사용설명서 (영문) |              |              |
| 서식번호           | DM-09-5            | Revised Date | Explications |
| 치수             | 155X175            | 2011. 05. 23 | DM-09-5 주소변경 |
| 용지재질           | 모조지                |              |              |
| 작업자            | 홍선미                |              |              |
| Printing Color |                    |              |              |
|                |                    |              |              |
|                |                    |              |              |
|                |                    |              |              |
|                |                    |              |              |