

meta Surgical

Head Office

Meta Biomed Co.,Ltd.

270 Osongsaengmyeong1-ro,
Osong-eup, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do, 28161, Republic of Korea

Phone: +82 43 230 8847

Fax: +82 43 217 1988

medical@meta-biomed.com

www.meta-biomed.com

Meta Biomed Inc.

121 Independence Lane
Chalfont, PA 18914, USA

Phone: +1 267 282 5893

america@metabiomed-inc.com

www.buymetabiomedonline.com

Meta Biomed Europe GmbH

Schederhofstraße 59, 45145 Essen,
Germany


Phone: +49 208 30991910

info@metabiomed-europe.com

www.meta-europe.com

MetaClip-P & Appliers

Polymer Ligating Clips & Appliers

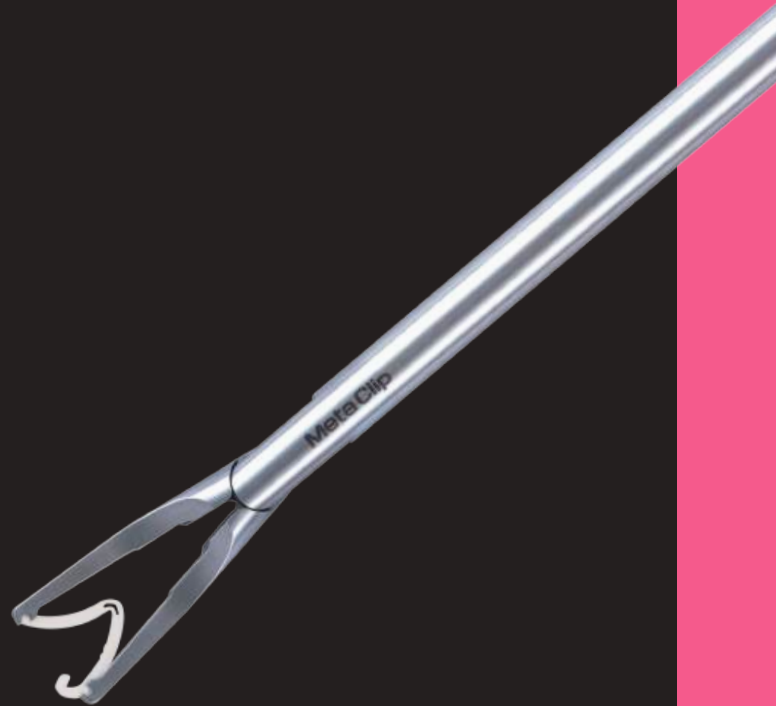



meta Surgical


MetaClip-P & Appliers

Polymer Ligating Clips & Appliers




– MetaClip-P Features

- Non slip type
- Clear Locking sound
- Biocompatible Polymer
- Non-absorbable Polymer
- Flexible Hinge
- High tensile strength

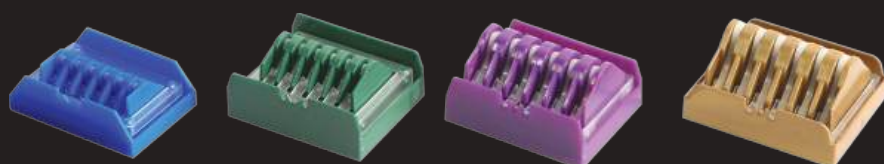

– Clip Specification

Vessels / Tissues bundle size range

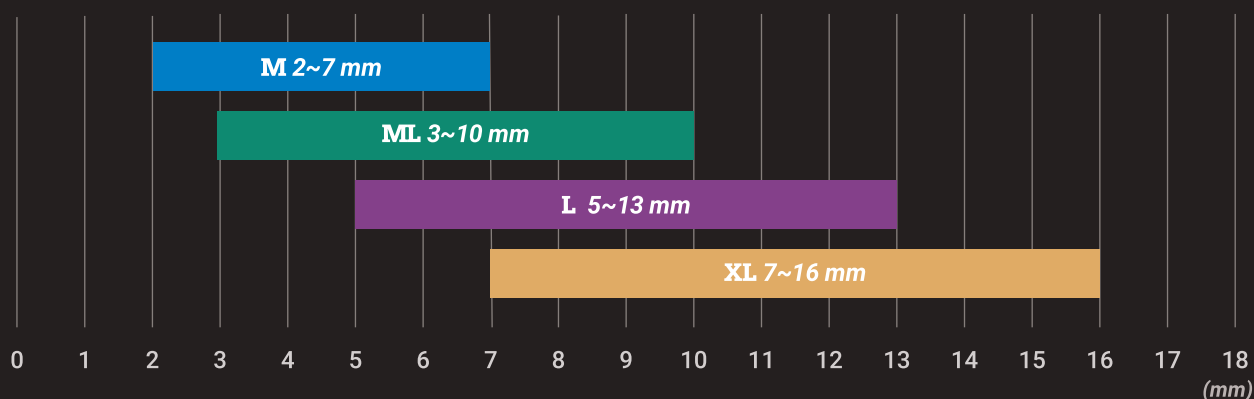


Clip	Ref No.	Clips	Cartridges / Box	Clips / Box
M	PCMM01A	6	14	84
ML	PCML02A	6	14	84
L	PCLL03A	6	14	84
XL	PCXL04A	6	14	84



CLIP

Non-Absorbable Polymer

is a biocompatible polymer that does not interfere with CT, MRI or X-ray.

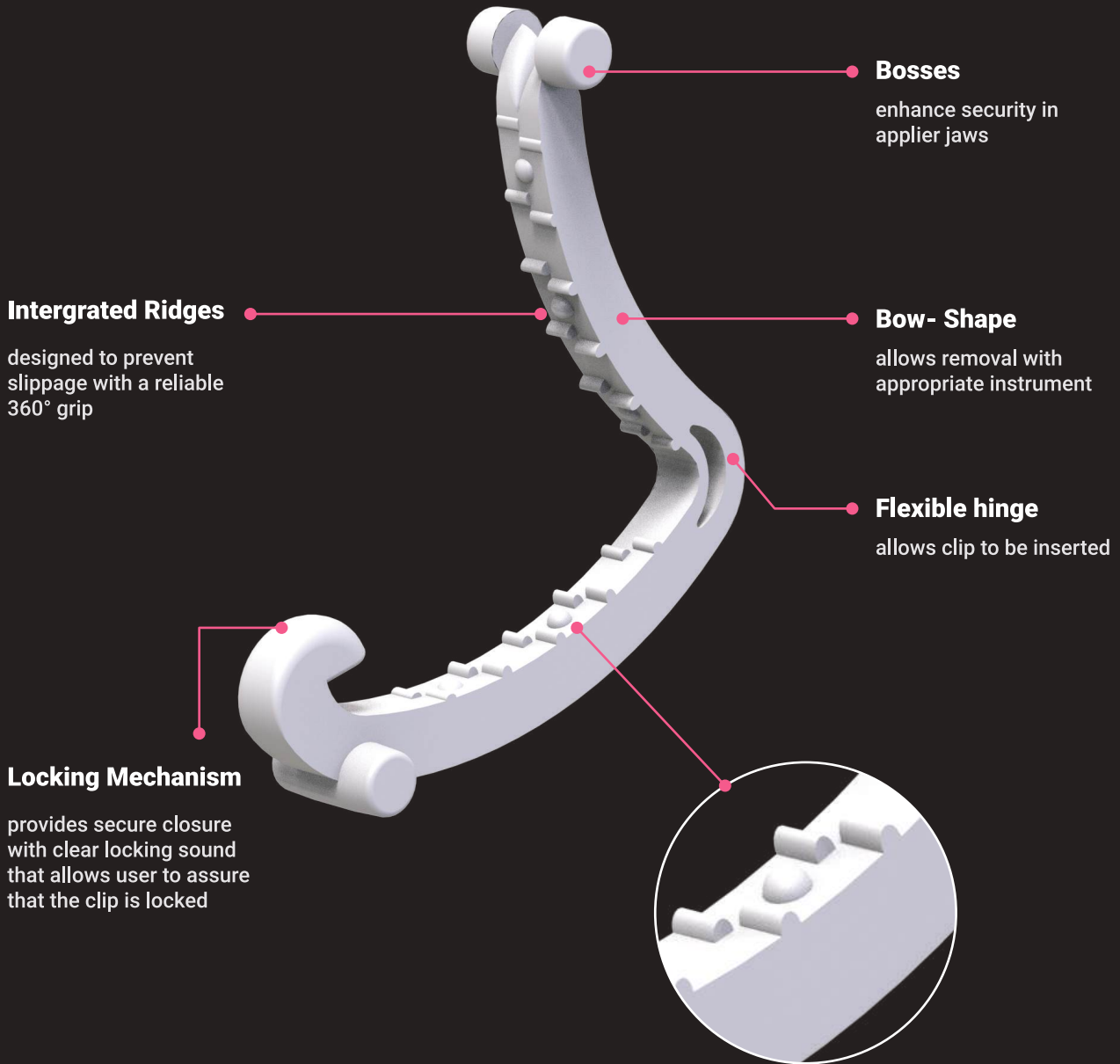

The teeth is designed to prevent slippage

The circular projection inside the clip prevents slippage

Appliers

Proper Jaw Angle

for better loading it is necessary to position the applier jaw perpendicular to the cartridge body



– Appliers Features

- Non - detachable design for easier maintenance and durability
- Made of high quality stainless steel
- Color coded size matching cartridge
- Compatible with corresponding sizes of Meta Clip-P polymer



– Applier Sterilization

○ Sterilization Conditions

Wrap with a disinfection Cloth.

○ Instrument State

Pre-vacuum

APPLIER	TEMPERATURE	EXPOSURE TIME	DRYING TIME
All Type Applier	134° C	3 minutes	30 minutes




◦ OPEN APPLIER



Clip	Model No	Length
M	OMAPMMA	280mm
ML	OMAPMLA	280mm
L	OMAPLLA	280mm
XL	OMAPXLA	280mm

◦ ENDO APPLIER

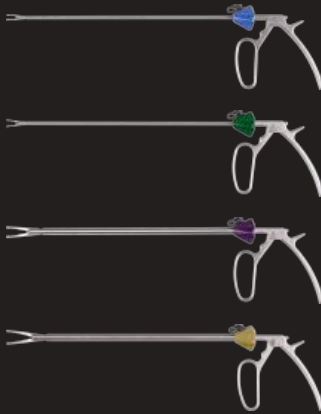


Clip	Model No	Shaft Length	Shaft Diameter
M	EMAPMMA	330mm	5mm
ML	EMAPMLA	330mm	5mm
L	EMAPLLA	330mm	10mm
XL	EMAPXLA	330mm	10mm

◦ OPEN REMOVER



Clip	Model No	Length
M/ML	ORAPMLA	280mm
L/XL	ORAPXLA	280mm

◦ ENDO REMOVER

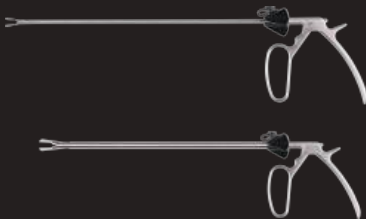

Clip	Model No	Shaft Length	Shaft Diameter
M/ML	ERAPMLA	320mm	5mm
L/XL	ERAPXLA	320mm	10mm



◦ Order Information

Clip	Open Applier	Endo Applier	Open Remover	Endo Remover
M	OMAPMMA	EMAPMMA	ORAPMLA	ERAPMLA
ML	OMAPMLA	EMAPMLA		
L	OMAPLLA	EMAPLLA	ORAPXLA	ERAPXLA
XL	OMAPXLA	EMAPXLA		

